



ProCair Plus Alarm Codes

The system has different alarm codes, each with a unique identifier that displays in the window

Press the Alarm Mute button to silence the audible alarm (for a maximum of 20 minutes), while the item is being assessed.



Code	Trigger	Solution
	Air Cells have failed to reach the pre-set pressure	<p>Ensure red CPR valve is closed</p> <p>Unzip and check cells – both clips need to be connected on each cell – if one found open, click back together</p> 
	Battery charge is below 10%	<p>Check the blue control unit is connected to mains power</p> 
	Control unit is unable to detect connection to mattress	<p>Ensure air hose is securely connected to control unit. Unclip and reconnect.</p> 
	The Bed Exit alarm has been turned on and the patient has gotten out of bed	<p>Press the Bed Exit button to turn off this function</p> 

To reset the system after troubleshooting
turn off the pump using the power button on the front of the pump
then wait 10 seconds ... and turn it back on

If you are unable to resolve the issue please call
H5 Healthcare 1300 727 631