





H5 Healthcare

Heelevate™

Instructions for Use

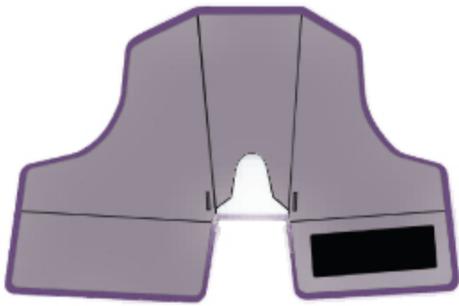


Place foot on open Heelevate™ with the ankle aligned to allow for a custom fit

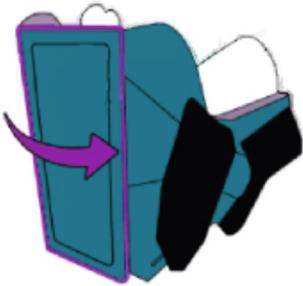

Fold up the sides of the Heelevate™ heel protector and close the footgate



Secure the flexible straps across the leg and foot
Position the anti-foot drop straps to keep the foot at a 90 degree angle

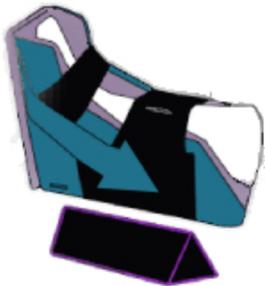

Attach the wedge to the side of the heel protector to assist with preventing hip and leg rotation

Product Description

The Heelevate™ offloading device is designed to

- help prevent pressure injuries
- minimise the risk of plantarflexion contractures
- support the ankle in a neutral position.

Cleaning Instructions

- Wipe clean using standard hospital disinfectant wipes that are bleach-free

Caution

- Do not walk or stand while wearing this device
- Intended for use by a single patient; may be reused by that patient

Size	Calf Size	Product Code
Small	17-32 cm	HESML
Standard	33-45 cm	HESTD
Maxi	46-64 cm	HEMAX